SISTEMAS DE IMPLANTES
IMPLANT SYSTEMS

www.gmidental.com

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KEY FEATURES

- Surface treatment / studies
- Recommendations of use
- Labelling description

IMPLANT SYSTEMS

- **gmi® avantgard** dental implants
- Insertion sequence for **gmi® avantgard** implants
- **gmi® avantgard** implants range
- Drilling sequences for **gmi® avantgard** implants
- **gmi® avantgard PEAK** dental implants
- Insertion sequence for **gmi® avantgard PEAK** implants
- **gmi® avantgard PEAK** implants range
- Drilling sequences for **gmi® avantgard PEAK** implants

GMI® AVANTGARD ATTACHMENTS

- Healing abutments
- Implant carrier abutments
- Temporary cylinders
- Angled abutments
- Straight abutments
- Titanium base
- Multi-esthetic abutments
- Bioesthetic bases
- CrCo pre-milled blanks
- Replicas
- Impression coping
- Clinic and laboratory screws
- MEA and BE screws
- Recommended tightening torques

GMI® SURGICAL KITS

- **gmi® frontier/avantgard** surgical kit
- **gmi® avantgard** guided surgery kit
- **gmi® PEAK** surgical kit
- **gmi® avantgard PEAK** guided surgery kit

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- The **CE 2797** marking applies to medical devices certified through conformity assessment by a Notified Body under Regulation (EU) 2017/745.

- The **CE** marking (without an identification number) applies to self-certified medical devices under Regulation (EU) 2017/745.

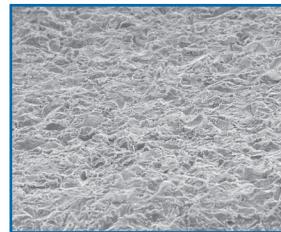
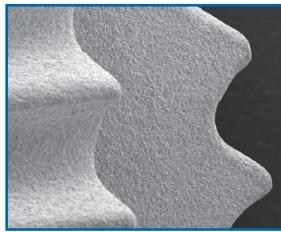
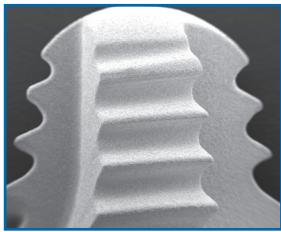
- Neither of these symbols applies to products that fall outside the scope of Regulation (EU) 2017/745 for medical devices.

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To promote the adhesion and growth of bone cells, the outer surface of the implant has been treated with the exclusive ADS® treatment (Advanced Double-Grip Surface) which combines a white corundum micro-bubble treatment and acid etching, for a non-smooth roughness that maximizes the contact surface between implant and bone and therefore improves stability and adhesion between them.

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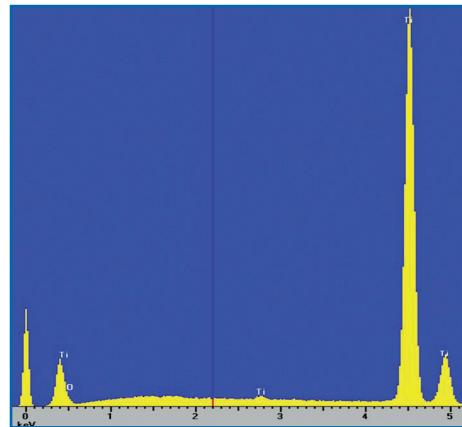
The ADS® treatment is the result of a series of collaborations with institutes and universities, as well as our own research and tests, that allow us to monitor and validate the optimal biological response to the implant surface: chemical (X-ray scattering studies to find out the composition of the implant and its surface) and topography studies (roughness studies) have been carried out, as well as biological (in vivo response studies in animals) and clinical studies.

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X-ray scattering studies show that the surface of the implants is only made of titanium and oxygen, which are the basic components of the oxide layer that is formed in the implant passivation process and that gives it its excellent corrosion resistance.

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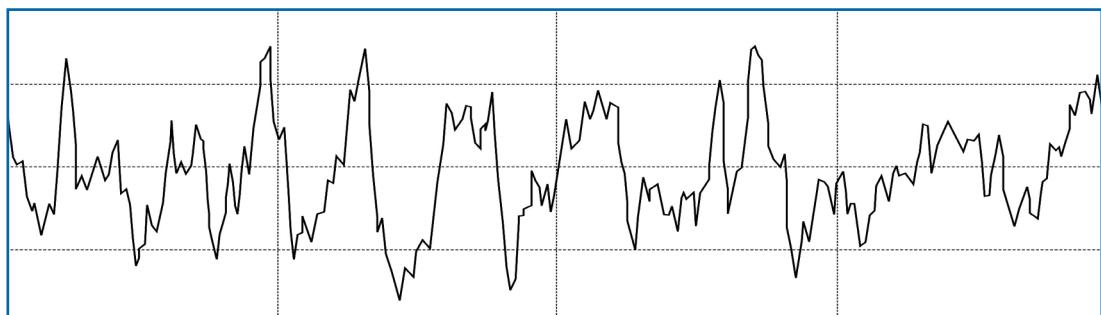
In order to control and identify the topography of the implant surface, roughness studies were performed with profilometer, as well as with techniques such as SEM (Scanning Electron Microscope) and CLSM (Confocal Laser Scanning Microscope), allowing to obtain and control the mean roughness (R_a) values within the parameters recommended in international publications.




Perfil de rugosidad obtenido con HOMMEL-WERKE T1000 • Roughness profile obtained with HOMMEL-WERKE T1000.

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
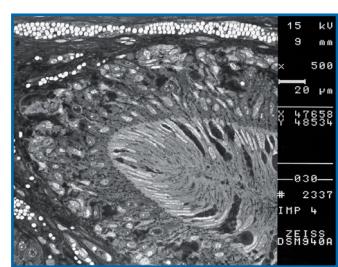
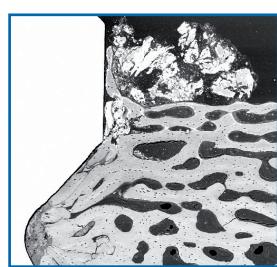
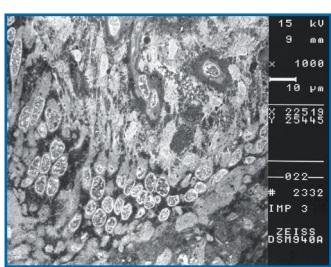
To determine the biological response of gmi implants, in vivo studies were conducted by inserting the implants on animals, leaving them without load during the healing process and performing a histological study to prove the excellent response of the bone cells and the complete osseointegration of the implant.

SEM-BSE: detalles ultraestructurales de las células en biofilm sobre el implante. • SEM-BSE: Ultrastructural details of cells in biofilm on the implant.

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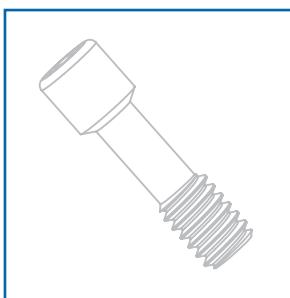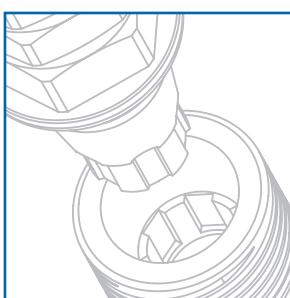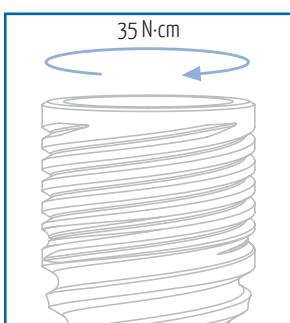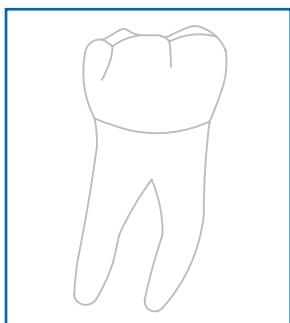The gmi® **avantgard** implant range was designed and tested to be inserted as a single restoration, with a 30-degree maximum inclination to the perpendicular plane to the occlusal plane, in accordance with the following diagram.

ø3,30 - ø3,75 mm	ø4,25 mm	ø4,75 - ø5,75 mm
Incisivos (Excepto centrales maxilares) Incisors (Except central maxillary incisors)	Caninos • Canines Premolares • Premolars Incisivos centrales maxilares Central maxillary incisors	Molares Molars

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The gmi® **avantgard** implant range is designed to be inserted applying a 35-40 N·cm torque, although it can bear a maximum tightening torque of 60 N·cm. gmi® recommends manual insertion of the implant with a dynamometric ratchet wrench to ensure that these values are not exceeded.

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The gmi® **avantgard** range of implants and prosthetic attachments are designed and manufactured with tolerances intended to maintain adjustments that minimize the clearance between components and therefore guarantee the stability of the connection. gmi® recommends the use of original attachments to ensure a perfect fit between the components of the restoration.

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
To prevent prior deformation of the clinic screw, gmi® recommends restricting its use only for the final tightening of the prosthesis. To make adjustments during the prosthetic procedures, use of the laboratory screw is recommended.

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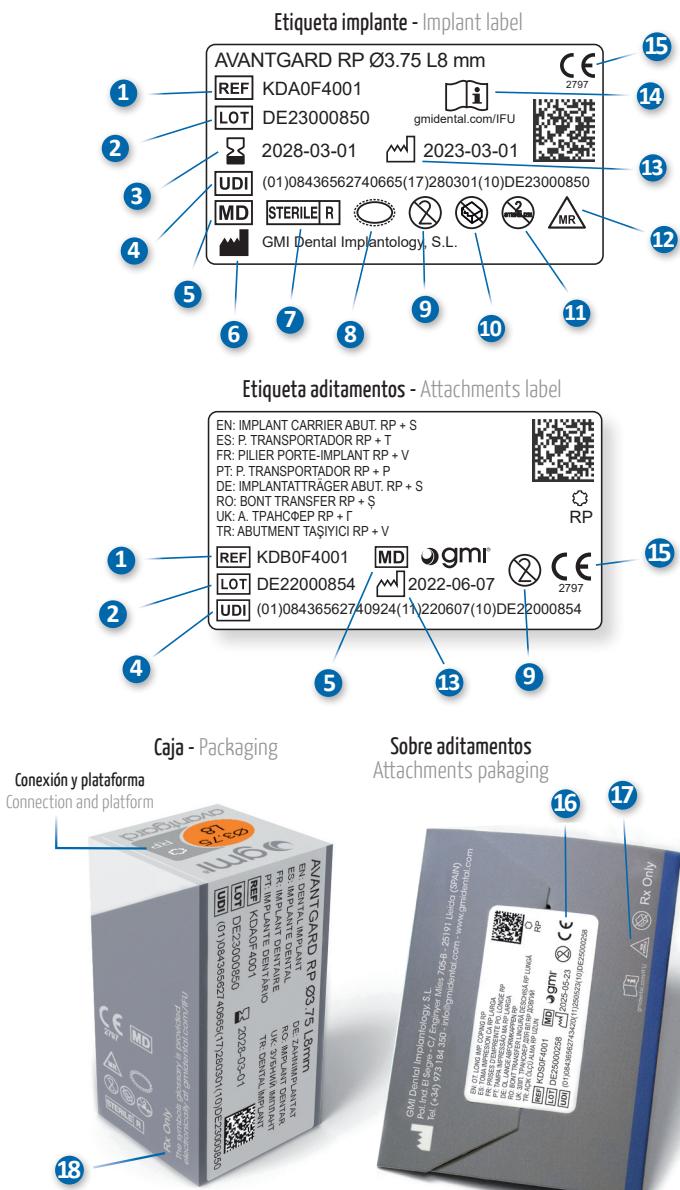
In the chart are detailed all the symbols appearing on the gmi® avantgard implant system's labelling and packaging and their corresponding description.

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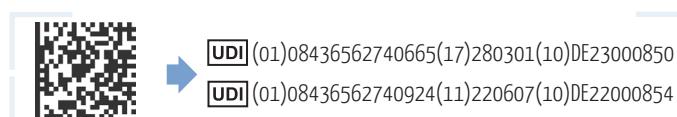*Minimum requirements for downloading and opening the electronic instructions for use:

- Stable internet connectivity.
- A device with a modern operating system that supports PDF viewing.
- Adequate storage space on your device.
- A web browser such as Google Chrome, Mozilla Firefox, Safari, Microsoft Edge or Opera installed on your device.
- A PDF reader application installed on your device.

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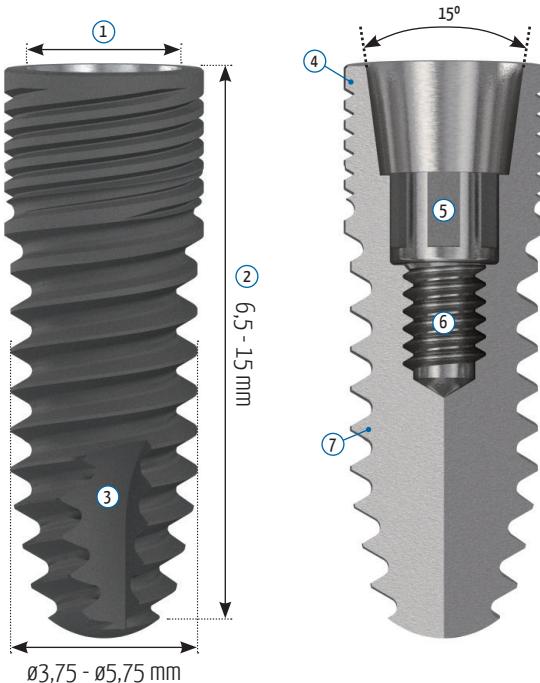
gmi® **avantgard** conical connection implants are machined in c.p. grade IV titanium of the highest quality, thus ensuring a high chemical stability and excellent biocompatibility. The titanium oxide layer, that is formed on the implant surface after the passivation process, and the careful design of the outer surface of the thread, guarantee good osseointegration and high primary stability.

The external dual inlet thread of the implant helps to reduce surgery time. Moreover, the rounded shapes, self-threading millings and slight tapering facilitate implant insertion, reducing tension at the bone-implant interface and thus preventing threading stress-related problems.

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③ Self threading millings

Variable length
3 millings at 120°
Atraumatic tip

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gmi® avantgard conical connection implants are packed in a screen-printed cardboard box, with the traceability labels and a double packaging containing the implant system, with the secondary packaging made of technical plastic and the primary packaging made of Pyrex glass to ensure the inert performance of the container. Once packaged in a clean room, they are sterilized with gamma radiation according to regulations.

The new packaging system of the gmi® avantgard implants consists of a titanium support which houses the dental implant, prepared to be inserted directly into the mouth, using the implant carrier wrench, designed for this purpose. This new system offers the odontologist the following advantages: more control and a better view of the implant insertion process, it is easier to use in reduced interdental spaces and a reduction in surgery time.*

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* Consult the instructions for use in electronic format.

1. Check sticker indicating implant diameter and length and open the carton.
2. Remove the secondary packaging and product identification labels.
3. Check the integrity of the safety seal and unscrew the plug on the secondary packaging.
4. Remove the primary packaging from the secondary packaging avoiding hitting it against a hard surface.
5. Extract and save the primary packaging plug making a lateral movement.
6. Insert the insertion wrench facing the wrench grooves of the wrench with those of the implant.
7. Check that the wrench is fully inserted and turn slightly while gently pulling on the assembly.
8. Place the implant in the bone bed by applying a torque of 35 N·cm until the treated area is at the subcrestal level. Remove the wrench by making an axial movement.
9. Remove the cover screw from the primary packaging plug, using the HEX-1.20 mm wrench.
10. Screw the cover screw to the implant by applying a maximum torque of 15 N·cm. Remove wrench and suture the incision.

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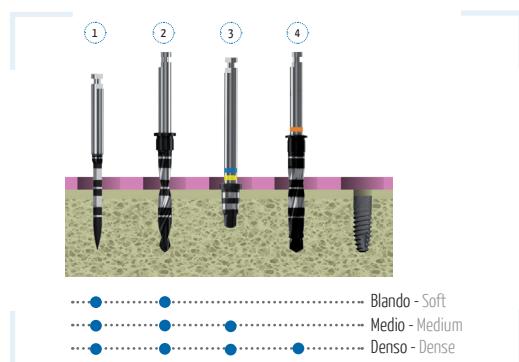
The following is a description of the drilling sequences for the different conical connection implant models, as well as the recommended drilling conditions for their use:


- Lance-shaped drill: 1200 - 1500 rpm.
- Pilot drill: 700 - 900 rpm.
- Step drills:
 - $\varnothing 3.00$ & $\varnothing 3.50$ mm → 400 - 700 rpm.
 - $\varnothing 4.00$ & $\varnothing 5.10$ mm → 400 - 600 rpm.
- Cortical drills: 200 - 400 rpm.

For drilling, external cooling with saline solution is required.

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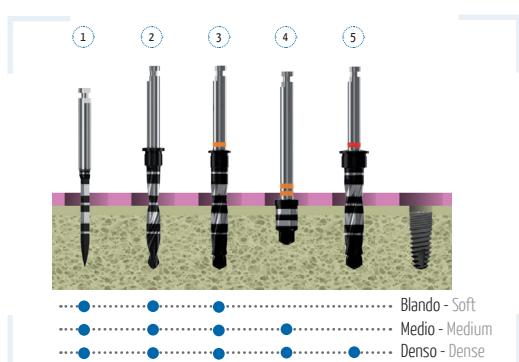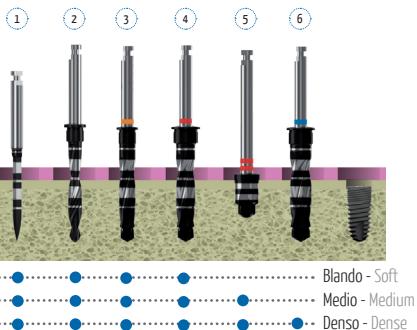
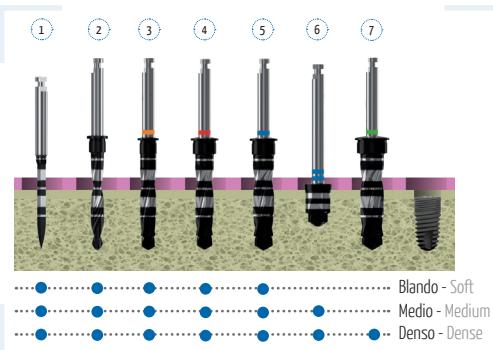
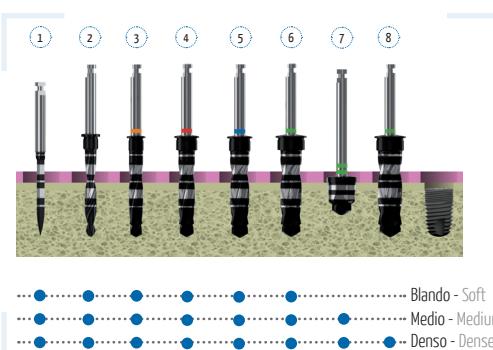
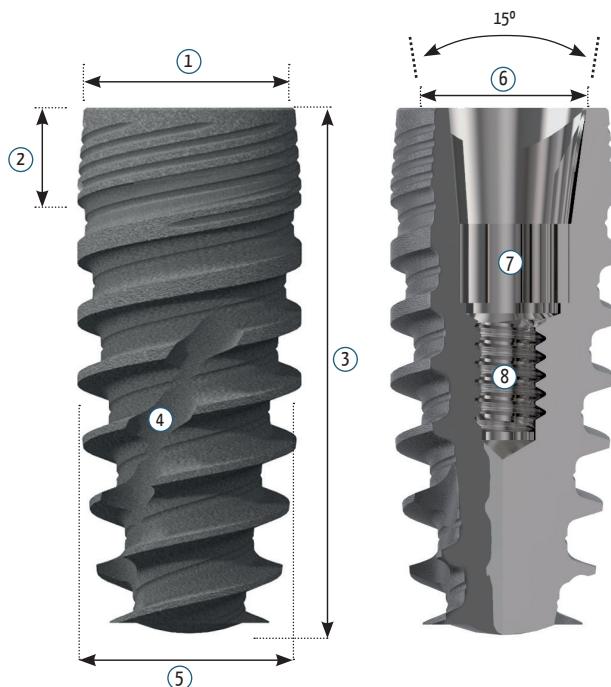
gmi® avantgard PEAK conical connection implants are machined in c.p. grade IV titanium of the highest quality, thus ensuring a high chemical stability and excellent biocompatibility. The titanium oxide layer, that is formed on the implant surface after the passivation process, and the careful design of the outer surface of the thread, guarantee good osseointegration and high stability.

The external dual inlet thread of the implant helps to reduce surgery time. Moreover, the conical core and progressive thread profile facilitate implant insertion and act as a bone compactor allowing high primary stability to be obtained in sparse bones.

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gmi® avantgard PEAK conical connection implants are packed in a screen-printed cardboard box, with the traceability labels and a double packaging containing the implant system, with the secondary packaging made of technical plastic and the primary packaging made of Pyrex glass to ensure the inert performance of the container. Once packaged in a clean room, they are sterilized with gamma radiation according to regulations.

The new packaging system of the gmi® avantgard PEAK implants consists of a titanium support which only houses the dental implant, prepared to be inserted directly into the mouth, using a single Avantgard NP-RP implant carrier wrench, designed for this purpose. This new system offers the odontologist the following advantages: more control and a better view of the implant insertion process, it is easier to use in reduced interdental spaces and a reduction in surgery time.*

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* Consult the instructions for use in electronic format.

1. Check sticker indicating implant diameter and length and open the carton.
2. Remove the secondary packaging and product identification labels.
3. Check the integrity of the safety seal and unscrew the plug on the secondary packaging.
4. Remove the primary packaging from the secondary packaging avoiding hitting it against a hard surface.
5. Extract and save the primary packaging plug making a lateral movement.
6. Insert the NP-RP insertion wrench facing the grooves of the wrench with those of the implant.
7. Check that the wrench is fully inserted and turn slightly while gently pulling on the assembly.
8. Place the implant in the bone bed by applying a torque of 35 N·cm until the treated area is at the subcrestal level. Remove the wrench by making an axial movement.
9. Remove the cover screw from the primary packaging plug, using the HEX-1.20 mm wrench.
10. Screw the cover screw to the implant by applying a maximum torque of 15 N·cm. Remove wrench and suture the incision.

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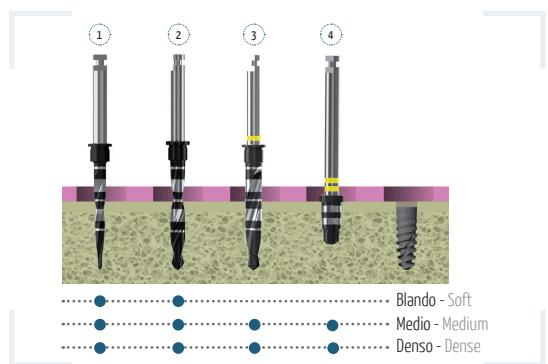
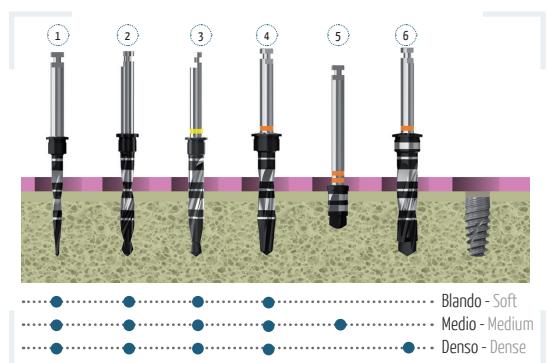
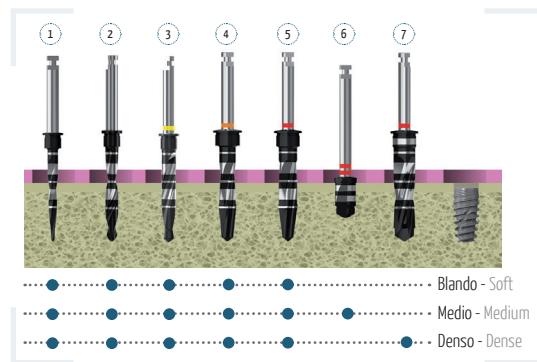
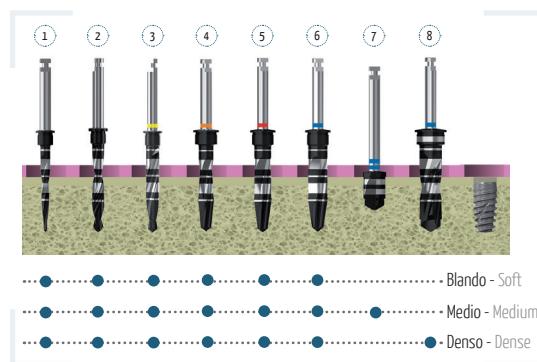
Once the repair phase of the tissue supporting the implant has finished, in the case of implants fully or partially covered by soft tissue there must be a mucosal channel or tunnel enabling connection of the implant to the prosthesis or secondary structure. Healing abutments are used to form the mucosal route, once the soft tissue has been opened by incision and the cover screw has been withdrawn, by screwing them to the implant.

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Key features

- Grade V titanium machining as per ISO-5832 standard.
- Tightening with b/f 1.20 mm hex wrench.
- Available in emergency heights from 1.50 to 4.50 mm.
- Apical bit to facilitate the start of threading.
- Manual tightening torque: Max. 15 N·cm.

Plataforma Platform	Ø cabeza Ø head	Altura (mm) Height (mm)	Ref.
NP ●●	Ø 4,00	1,50	KDPOC4012
		2,50	KDPOC4013
		3,50	KDPOC4014
		4,50	KDPOC4015
RP ●●	Ø 5,00	1,50	KDPOC4016
		2,50	KDPOC4017
		3,50	KDPOC4018
		4,50	KDPOC4019

Plataforma Platform	Ø cabeza Ø head	Altura (mm) Height (mm)	Ref.
RP ●●	Ø 5,00	1,50	KDPOC4001
		2,50	KDPOC4002
		3,50	KDPOC4003
		4,50	KDPOC4004
RP ●●	Ø 6,00	2,50	KDPOC4009
		3,50	KDPOC4010
		4,50	KDPOC4011

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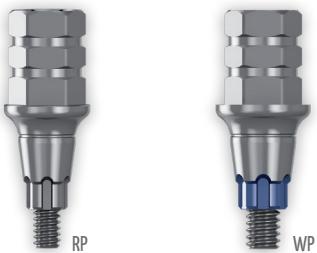
When the use of a custom abutment is required, implant carrier abutments can be used. By cutting them in any direction, they can be suited to the needs of the prosthetic. In addition and as a complementary function, implant carrier abutments can be used like closed tray impression coping.

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Key features

- Grade V titanium machining as per ISO-5832 standard.
- They allow cutting in any direction to suit the needs of the prosthetic.
- Compatible with b/f 3.00 mm hex wrench for use as an implant carrier.
- Available only in non-rotary version.
- Possibility to use as impression coping.
- Premounted clinic screw included.
- Screw tightening torque: 25 N·cm.

Plataforma Platform	Ø hombro Ø shoulder	Ref.
RP ●●	Ø 3,75 - Ø 4,25	KDBOF4001
	Ø 4,75	KDBOF4002
WP ●	Ø 5,75	KDBOF4003

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When it is necessary to perform a provisional restoration while the bone-implant interfaces or the soft tissues are healing, the temporary abutment is used, defined as attachment that once attached to the crown of the implant with clinic screw, they will allow a temporary restoration to be carried out while the definitive restoration is being manufactured.

Key features

- Machined in high strength technical plastic.
- NP abutments machined in Grade V titanium as per ISO-5832 standard
- Rotary version only for multiple restorations.
- Anti-rotational plane and transverse retention to provide good fixation.
- Clinic screw included.
- Manual tightening torque: Max. 15 N·cm.

Plataforma Platform	Tipo conexión Connection type	Ref.
RP ●●	Anti-rotatoria • Non-rotary	KDHoF4001
	Rotatoria • Rotary	KDHoF4002
WP ●	Anti-rotatoria • Non-rotary	KDHoF4003
	Rotatoria • Rotary	KDHoF4004

Plataforma Platform	Altura (mm) Height (mm)	Tipo conexión Connection type	Ref.
NP ●●	1,50	Anti-rotatoria • Non-rotary	KDHoF4009
	3,00	Anti-rotatoria • Non-rotary	KDHoF4010
	1,50	Rotatoria • Rotary	KDHoF4011
	3,00	Rotatoria • Rotary	KDHoF4012

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When it is necessary to correct an extreme angular position of the implant inside the bone structure in relation to adjacent natural or prosthetic teeth, angled abutments should be used. Once mounted and oriented on the implant, these abutments will allow the parallelization of the new crown with other structures.

Key features

- Grade V titanium machining as per ISO-5832 standard.
- Allow non-parallelism correction up to 20 degrees.
- Clinic screw included.
- Abutment shoulder adapted to the shape of the gum.
- Screw tightening torque: 25 N·cm.

Plataforma Platform	Ang. (º) Angle (°)	Altura (mm) Height (mm)	Ref.
NP ● ●	10º	2,50	KDMoF4039
	10º	4,50	KDMoF4040
	20º	2,50	KDMoF4041
	20º	4,50	KDMoF4042

Plataforma Platform	Ang. (º) Angle (°)	Altura (mm) Height (mm)	Ref.
RP ● ●	15º	2,50	KDMoF4013
	15º	4,50	KDMoF4014
	20º	2,50	KDMoF4015
	20º	4,50	KDMoF4016

Plataforma Platform	Ang. (º) Angle (°)	Altura (mm) Height (mm)	Ref.
WP ●	15º	2,50	KDMoF4017
	15º	4,50	KDMoF4018
	20º	2,50	KDMoF4019
	20º	4,50	KDMoF4020

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Once the mucosal tunnel has been formed by the healing abutment and after removing it from this location, it is appropriate to use the abutment, which is defined as the machined tapered attachment that, after being fixed with the clinic screw to the top of the implant, acts as a support for the clinic crown and is attached to it using cement.

Key features

- Grade V titanium machining as per ISO-5832 standard.
- Anti-rotational planes to increase retention.
- Available only in non-rotary version.
- Clinic screw included.
- Available in four emergency heights from 1.50 to 4.50 mm.
- Screw tightening torque: 25 N·cm.

Plataforma Platform	Ø hombro Ø shoulder	Altura (mm) Height (mm)	Ref.
NP ● ●	Ø 4,00	1,50	KDMoF4035
		2,50	KDMoF4036
		3,50	KDMoF4037
		4,50	KDMoF4038

Plataforma Platform	Ø hombro Ø shoulder	Altura (mm) Height (mm)	Ref.
RP ● ●	Ø 4,00	1,50	KDMoF4001
		2,50	KDMoF4002
		3,50	KDMoF4003
		4,50	KDMoF4004
RP ● ●	Ø 5,00	1,50	KDMoF4005
		2,50	KDMoF4006
		3,50	KDMoF4007
		4,50	KDMoF4008

Plataforma Platform	Ø hombro Ø shoulder	Altura (mm) Height (mm)	Ref.
WP ●	Ø 6,00	1,50	KDMoF4009
		2,50	KDMoF4010
		3,50	KDMoF4011
		4,50	KDMoF4012

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Once the mucosal tunnel has been formed by the healing abutment and after removing the cap from this location, it is appropriate to use the titanium base, which is defined as the machined tapered attachment that, after being fixed with the clinic screw to the top of the implant, acts as a support for a metallic or ceramic meso-structure casted or machined with CAD/CAM.

Key features

- Grade V titanium machining as per ISO-5832 standard.
- Non-rotary and rotary version.
- Clinic screw included.
- Available in emergency height of 1.50 mm and 3.00 mm.
- Non-rotary titanium base with indexation.
- Screw tightening torque: 25 N·cm.
- Scan directly on Titanium base or alternative scan on Scan abutment.


Base titanio - Titanium base			
Plataforma Platform	Altura (mm) Height (mm)	Tipo conexión Connection type	Ref.
NP ● ●	1,50	Anti-rotatoria • Non-rotary	KDMoF4043
	1,50	Rotatoria • Rotary	KDMoF4045
	3,00	Anti-rotatoria • Non-rotary	KDMoF4044
	3,00	Rotatoria • Rotary	KDMoF4046
RP ● ●	1,50	Anti-rotatoria • Non-rotary	KDMoF4021
	1,50	Rotatoria • Rotary	KDMoF4023
	3,00	Anti-rotatoria • Non-rotary	KDMoF4025
	3,00	Rotatoria • Rotary	KDMoF4027
WP ●	1,50	Anti-rotatoria • Non-rotary	KDMoF4022
	1,50	Rotatoria • Rotary	KDMoF4024
	3,00	Anti-rotatoria • Non-rotary	KDMoF4026
	3,00	Rotatoria • Rotary	KDMoF4028

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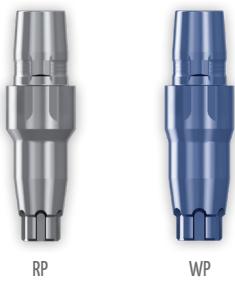Following the first stage of surgery and therefore after implant osseointegration is achieved, the multi-esthetic abutment (MEA) is used, that is defined as the machined attachment that, after being attached to the crown of the implant, serves as a support for a multiple restoration, thus decreasing the height between the implant connection and the structure connection, and providing a passive adjustment of the structure.

Key features

- Grade V titanium machining as per ISO-5832 standard.
- Available in emergency heights from 1.50 to 4.50 mm.
- One-piece straight abutments in rotary version.
- Tightened with hex wrench: B/F 2.00 mm (KYLoCo149).
- Impression coping with screw included.
- Castable with clinic screw included.
- Temporary cylinder in grade V titanium with clinic screw included.
- Angled multi-esthetic abutments allow non-parallelism up to 30°.
- Tightening torques: ME straight abut. (30 N·cm), ME angled abut. (20 N·cm) and healing caps (Manual: Max. 15 N·cm).

NP

RP

WP

Pilares multi-estético rectos Multi-esthetic straight abutments

Plataforma Platform	Altura (mm) Height (mm)	Ref.
NP	1,50	KDGOF4019
	2,50	KDGOF4020
	3,50	KDGOF4021
	4,50	KDGOF4022
RP	1,50	KDGOF4001
	2,50	KDGOF4002
	3,50	KDGOF4003
	4,50	KDGOF4004
WP	1,50	KDGOF4005
	2,50	KDGOF4006
	3,50	KDGOF4007
	4,50	KDGOF4008

Pilares multi-estético gmi® avantgard - gmi® avantgard multi-esthetic abutments

Pilares multi-estético inclinados Multi-esthetic angled abutments

Plataforma Platform	Ang. (º) Angle (°)	Altura (mm) Height (mm)	Ref.
RP ●●	17º	2,50	KDGoF4009
	17º	3,50	KDGoF4010
	30º	4,00	KDGoF4011


Pilares cicatrización PME Healing abutments MEA

Plataforma Platform	Ø Diámetro (mm) Ø Diameter (mm)	Ref.
NP - RP - WP ●●	Ø 4,80	KDpOf0035
	Ø 6,00	KDpOf0037

Tomas impresión cubeta abierta (CA) larga PME Open tray (OT) long impression coping MEA

Plataforma Platform	Ref.
NP - RP - WP ●●	KDSOf0015

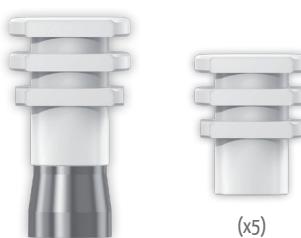Tomas impresión cubeta cerrada (CC) PME Closed tray (CT) impression coping MEA

Plataforma Platform	Cantidad (Ud.) Quantity (Un.)	Ref.
NP - RP - WP ●●	1	KDSOf0017
	5	KDSOf5001

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Once the insertion of the implant in the bone bed has been completed, the bioesthetic base is used, which is defined as the machined attachment that, once fixed to the coronal part of the implant, allows single restorations at the abutment level.

Placing the bioesthetic base during surgery allows the soft tissues to heal around it.

Once the tissue has healed, it serves as the basis for the rehabilitation, avoiding the connection and disconnection of the components to the implant and, therefore, keeping the biological seal intact.

Key features

- Grade V titanium machining as per ISO-5832 standard.
- Available in emergency heights from 2.50 to 4.50 mm.
- Straight two-piece bases, including the screw.
- Tightened with hex wrench: B/F 2.00 mm (KYLoC0149).
- Impression coping with screw included.
- Castable.
- Tightening torques: BE base (25 N·cm) and healing abutment (Manual: Max. 15 N·cm).

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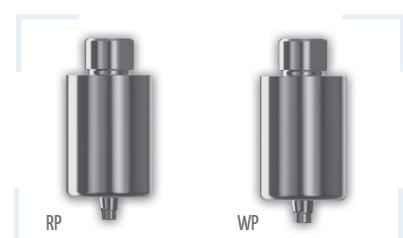
CrCo pre-milled blanks consist of a block with the implant connection machined, which allows an excellent fit between the implant and the abutment.

They are indicated to produce custom one-piece abutments using CAD/CAM allowing direct application of ceramic.

Plataforma Platform		Ref.
RP ●●	10	TS00005
WP ●	10	TS00006

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Once the transfer of the position has been made using the impression coping from the biological medium to the laboratory working model, it is necessary to use replicas, which are defined as the machined attachments designed on one hand to reproduce the implant position in the mouth of the patient to a working model, and on the other to serve as a connection model for the construction in the laboratory of the structure to carry out the restoration.

Key features

- Grade V titanium machining as per ISO-5832 standard.
- Anti-rotation millings and vertical retention neck for a good fixation.

Réplicas de implante Implant replicas	
Plataforma Platform	Ref.
NP ●●	KDRoC4005
RP ●●	KDRoC4001
WP ●	KDRoC4002

Réplicas digitales Digital replicas	
Plataforma Platform	Ref.
NP ●●	KDRoF4006
RP ●●	KDRoF4003
WP ●	KDRoF4004

Réplicas para pilares multi-estético MEA replicas	
Plataforma Platform	Ref.
NP - RP - WP ●●	KDRoC0012

Réplicas digitales para pilares multi-estético MEA digital replicas	
Plataforma Platform	Ref.
NP - RP - WP ●●	KDRoF0014

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Impression coping are defined as the machined attachments that, connecting to the coronal part of the implant within the oral cavity and axially fixed by the impression screw, is used to make the transfer of the position of the implant in the biological medium to a laboratory model, on which the prosthetic will work to create the dental crowns or prostheses. This process is achieved using impression materials placed in an appropriate manner, that will harden in the oral cavity and produce the negative of the patient's mouth. Once extracted, replicas are placed and the casting plaster is poured to obtain the positive mould where the replica will be placed in the original position of the implant in the mouth.

Key features

- Machined open tray impression coping in grade V titanium according to ISO-5832.
- Impression screws in grade V titanium according to ISO-5832.
- Tightening of impression screws with b/f 1.20 mm hex wrench.
- Apical bit to facilitate the start of threading impression screw.
- Anti-rotation millings and vertical retention neck for a good fixation.
- Impression coping screw included.
- Long impression screw with longitudinal millings for manual tightening.
- CT impression coping machined in technical plastic.
- Manual tightening torque: Max. 15 N·cm.

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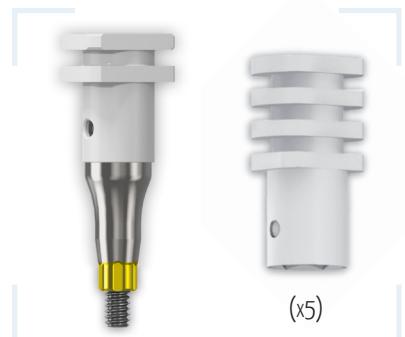
Clinic screws are defined as machined devices used for the final anchoring of the abutments or the castable directly to the implant, while laboratory screws are the machined devices used for the temporary anchoring of abutments to the replica of the implant used by the dental laboratory where the construction of dental crowns or prostheses takes place.

Key features

- Machined grade V titanium screws according to ISO-5832.
- Tightening of screws with b/f 1.20 mm hex wrench.
- TX screw allows the screw channel to be angled up to 20°.

Tornillo clínica Clinic screw	
Plataforma Platform	Hex. llave (mm) Hex. wrench (mm)
NP - RP ●●	○ 1,20 mm KDTOC4003
	∅ TX KDTOC4021
WP ●	○ 1,20 mm KDTOC4004
	∅ TX KDTOC4022

Tornillo laboratorio Laboratory screw	
Plataforma Platform	Hex. llave (mm) Hex. wrench (mm)
NP - RP ●●	○ 1,20 mm KDTOC4005
	∅ TX KDTOC4023
WP ●	○ 1,20 mm KDTOC4006
	∅ TX KDTOC4024

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MEA and BE clinical screws are defined as machined devices used for the final anchoring of the ME and BE abutments, while laboratory screws are the machined devices used for the temporary anchoring of ME and BE abutments to the replica during the fabrication of the prostheses in the laboratory.

Key features

- Machined grade V titanium screws according to ISO-5832.
- Tightening of screws with b/f 1.20 mm and 2.00 mm hex wrench.
- TX screw allows the screw channel to be angled up to 20°.

Tornillo PME MEA screw		
Plataforma Platform	Hex. llave (mm) Hex. wrench (mm)	Ref.
NP - RP - WP ● ●	○ 1,20 mm ∅ TX	KDToC0056 KDToC0070

Tornillo laboratorio PME MEA laboratory screw		
Plataforma Platform	Hex. llave (mm) Hex. wrench (mm)	Ref.
NP - RP - WP ● ●	○ 1,20 mm ∅ TX	KDToC0057 KDToC0072

Tornillo BE BE screw		
Plataforma Platform	Hex. llave (mm) Hex. wrench (mm)	Ref.
RP - WP ● ●	○ 1,20 mm ∅ TX	KDToC4016 KDToC0074

Tornillo laboratorio BE BE laboratory screw		
Plataforma Platform	Hex. llave (mm) Hex. wrench (mm)	Ref.
RP - WP ● ●	○ 1,20 mm ∅ TX	KDToC4020 KDToC0075

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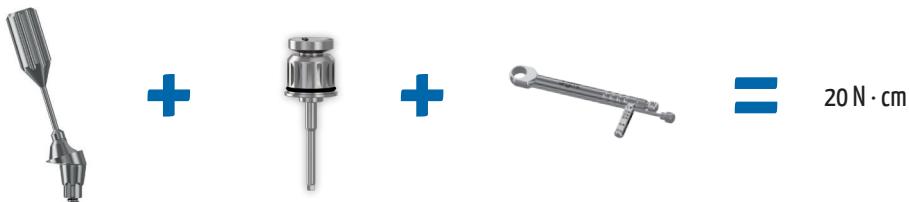
gmi® frontier/avantgard surgical kit consists of an autoclavable case made of high strength resin, containing all the necessary material for the correct positioning of gmi frontier and avantgard dental implant systems. The components are arranged in such a way to be easy to find in each moment and in any stage of the intervention.

gmi® surgical kits are distributed in boxes made of high strength technical polymer, and must be sterilized in a moist heat autoclave before use at a maximum temperature of 134°C.

To complement this kit, gmi® has the PEAK drill Kit (*see instrumental catalogue*), allowing it to be easily expanded for the placement of 2 additional implant systems (frontier PEAK and avantgard PEAK).

	Descripción - Description	Un.	Ref.
Fresa lanza	Lance-shaped drill	1	KYFOC1225
Fresa piloto ø2,50 mm	Pilot drill ø2.50 mm	1	KYFOC2250
Fresa escalonada ø2,50 / ø3,00 mm	Step drill ø2.50 / ø3.00 mm	1	KYFOC5130
Fresa escalonada ø3,00 / ø3,50 mm	Step drill ø3.00 / ø3.50 mm	1	KYFOC5135
Fresa escalonada ø3,50 / ø4,00 mm	Step drill ø3.50 / ø4.00 mm	1	KYFOC5140
Fresa escalonada ø4,00 / ø4,50 mm	Step drill ø4.00 / ø4.50 mm	1	KYFOC5145
Fresa cortical ø3,65 mm	Cortical drill ø3.65 mm	1	KYFOC4336
Fresa cortical ø3,20 mm	Cortical drill ø3.20 mm	1	KYFOC4332
Fresa cortical ø4,00 mm	Cortical drill ø4.00 mm	1	KYFOC4340
Fresa cortical ø4,50 mm	Cortical drill ø4.50 mm	1	KYFOC4345
Fresa cortical ø4,90 mm	Cortical drill ø4.90 mm	1	KYFOC4349
Fresa cortical ø5,90 mm	Cortical drill ø5.90 mm	1	KYFOC4359
Llave carraca IP	TI ratchet wrench	1	KYLOF0113
Llave-transportador HEX-2,45 mm IP corta	Short HEX-2.45 mm implant carrier TI wrench	1	KYLOF0096
Llave-transportador HEX-2,45 mm IP larga	Long HEX-2.45 mm implant carrier TI wrench	1	KYLOF0111
Llave-transportador HEX-2,45 mm CA	HEX-2.45 mm implant carrier HP wrench	1	KYLOF0091
Llave Avantgard NP/RP IP corta	Short Avantgard NP/RP TI wrench	1	KYLOF0157
Llave Avantgard NP/RP IP larga	Long Avantgard NP/RP TI wrench	1	KYLOF0162
Llave Avantgard WP IP corta	Short Avantgard WP TI wrench	1	KYLOF0168
Llave Avantgard WP IP larga	Long Avantgard WP TI wrench	1	KYLOF0169
Llave Avantgard NP/RP CA corta	Short Avantgard NP/RP HP wrench	1	KYLOF0151
Llave Avantgard WP CA corta	Short Avantgard WP HP wrench	1	KYLOF0166
Llave carraca corta HEX-1,20 mm	Short HEX-1.20 mm ratchet wrench	1	KYLOF0128
Llave carraca larga HEX-1,20 mm	Long HEX-1.20 mm ratchet wrench	1	KYLOF0129
Alargador fresas	Drill extender	1	KYLOC0023
Paralelizador ø2,00 / ø2,80 mm	Paralleling pin ø2.00 / ø2.80 mm	3	KYLOC0078
Indicador profundidad ø2,50 mm	Depth gauge ø2.50 mm	1	KYLOC0165

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gmi® avantgard guided surgery kit consists of an autoclavable case made of high strength resin, containing all the necessary material for performing totally guided surgeries with gmi® avantgard implants of ø3.30 / ø3.75 / ø4.25 and ø4.75 mm and lengths between 8 and 13 mm.

To complement this kit, gmi® has the **PEAK guided surgery drill Kit** (*see instrumental catalogue*), allowing it to be easily expanded for the placement of 1 additional implant system (avantgard PEAK).

Descripción - Description	Un.	Ref.	Descripción - Description	Un.	Ref.
Fresa Pin ø1,50 mm	1	KYFOC6060	Fresa escalonada DLC ø4/ø4.5 L13 mm	1	KYFOC6524
Trepante gingival CA ø5,10 mm	1	KYFOC6066	Fresa cortical DLC ø3,2 mm	1	KYFOC6401
Fresa inicial ø2,5 mm	1	KYFOC6101	Fresa cortical DLC ø3,55 mm	1	KYFOC6402
Fresa piloto DLC ø2,5 L8 mm	1	KYFOC6201	Fresa cortical DLC ø3,9 mm	1	KYFOC6403
Fresa piloto DLC ø2,5 L10 mm	1	KYFOC6202	Fresa cortical DLC ø4,4 mm	1	KYFOC6404
Fresa piloto DLC ø2,5 L11,5 mm	1	KYFOC6203	Fresa cortical DLC ø4,9 mm	1	KYFOC6405
Fresa piloto DLC ø2,5 L13 mm	1	KYFOC6204	Llave carraca corta Hex-1,20 mm	1	KYLOF0128
Fresa piloto DLC ø2,5 L15 mm	1	KYFOC6205	Adaptador Hex-4,00 mm LLCIP	1	KYLOC6054
Fresa escalonada DLC ø2,5/ø3 L8 mm	1	KYFOC6506	Adaptador Hex-4,00 mm CA	1	KYLOC6055
Fresa escalonada DLC ø2,5/ø3 L10 mm	1	KYFOC6507	Extractor transportadores	1	KYLOC6115
Fresa escalonada DLC ø2,5/ø3 L11,5 mm	1	KYFOC6508	Transportadores NP/RP + tornillo	2	KYLOF6107
Fresa escalonada DLC ø2,5/ø3 L13 mm	1	KYFOC6509	Transportadores WP + tornillo	2	KYLOF6108
Fresa escalonada DLC ø2,5/ø3 L15 mm	1	KYFOC6510	Llave directa NP/RP CA	1	KYLOF6111
Fresa escalonada DLC ø3/ø3,5 L8 mm	1	KYFOC6511	Llave directa WP CA	1	KYLOF6112
Fresa escalonada DLC ø3/ø3,5 L10 mm	1	KYFOC6512	Llave directa NP/RP LLCIP	1	KYLOF6113
Fresa escalonada DLC ø3/ø3,5 L11,5 mm	1	KYFOC6513	Llave directa WP LLCIP	1	KYLOF6114
Fresa escalonada DLC ø3/ø3,5 L13 mm	1	KYFOC6514	Pin lateral férula	3	KYLOC6059
Fresa escalonada DLC ø3/ø3,5 L15 mm	1	KYFOC6515	Llave carraca IP	1	KYLOF6013
Fresa escalonada DLC ø3,5/ø4 L8 mm	1	KYFOC6516	Llave transportador plana 45°	1	KYLOC0093
Fresa escalonada DLC ø3,5/ø4 L10 mm	1	KYFOC6517			
Fresa escalonada DLC ø3,5/ø4 L11,5 mm	1	KYFOC6518			
Fresa escalonada DLC ø3,5/ø4 L13 mm	1	KYFOC6519			
Fresa escalonada DLC ø3,5/ø4 L15 mm	1	KYFOC6520			
Fresa escalonada DLC ø4/ø4,5 L8 mm	1	KYFOC6521			
Fresa escalonada DLC ø4/ø4,5 L10 mm	1	KYFOC6522			
Fresa escalonada DLC ø4/ø4,5 L11,5 mm	1	KYFOC6523			

*Todas las referencias se venden por separado como recambios
*All references are sold separately as spare parts

Accesorios - Accessories	Descripción - Description	Un.	Ref.
Casquillo pin lateral férula	Sleeve for lateral pin	5	KYLOF6061
Casquillo férula ø5,10 mm	Sleeve ø5,10 mm	5	KYLOF6102

Kit quirúrgico gmi® PEAK - gmi® PEAK surgical kit

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gmi® PEAK surgical kit consists of an autoclavable case made of high strength resin, containing all the necessary material for the correct positioning of frontier PEAK and avantgard PEAK dental implant systems. The components are arranged in such a way to be easy to find in each moment and in any stage of the intervention.

gmi® surgical kits are distributed in boxes made of high strength technical polymer, and must be sterilized in a moist heat autoclave before use at a maximum temperature of 134°C.

To complement this kit, gmi® has the **frontier/avantgard drill Kit** (*see instrumental catalogue*), allowing it to be easily expanded for the placement of 2 additional implant systems (frontier and avantgard).

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gmi® avantgard PEAK guided surgery kit consists of an autoclavable case made of high strength resin, containing all the necessary material for performing totally guided surgeries with gmi® avantgard PEAK implants of Ø3.30 / Ø3.75 / Ø4.25 and Ø4.75 mm and lengths between 8 and 13 mm.

To complement this kit, gmi® has the **frontier/avantgard guided surgery drill Kit** (*see instrumental catalogue*), allowing it to be easily expanded for the placement of 1 additional implant system (avantgard).

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