3shape

Instructions to import  
gmi libraries  
into 3shape  software

## Download software

---

From the computer connect to **www.gmidental.com**

Go to **PRODUCTS** menu, open the **CAD/CAM Library > Interfase** section and select **3Shape**.

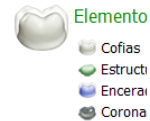
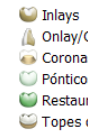
Click on **3shape**  icon to start the download.

A file called **3S\_GMI\_REVX.zip** (*where X is the updated revision number*) will be downloaded into *downloads folder* of your computer.

## Import library


The library is imported using the *Control Panel* of 3shape.

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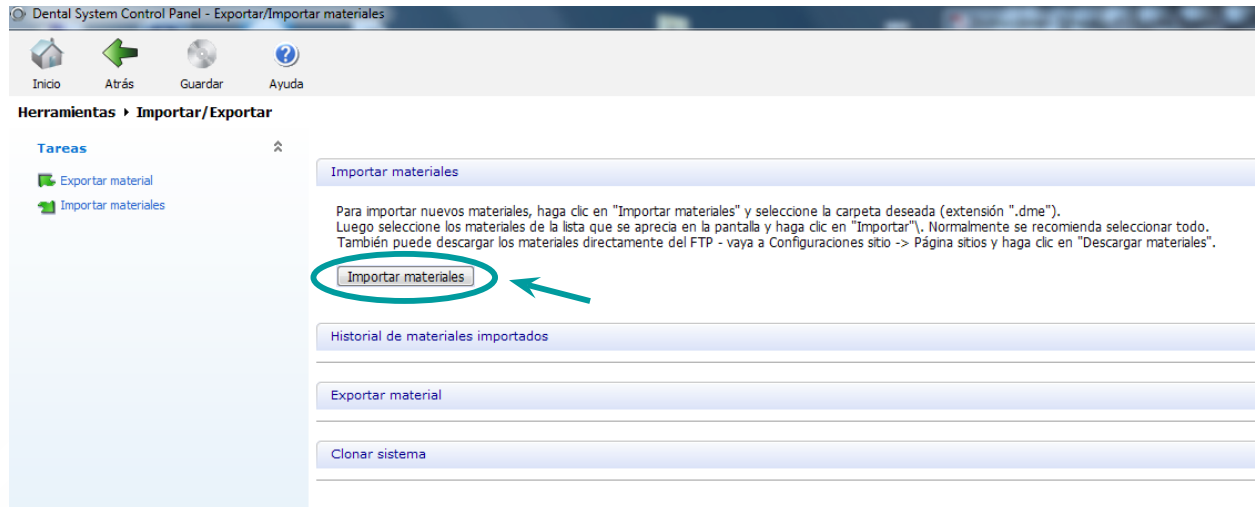
Clonar sistema

## Import library

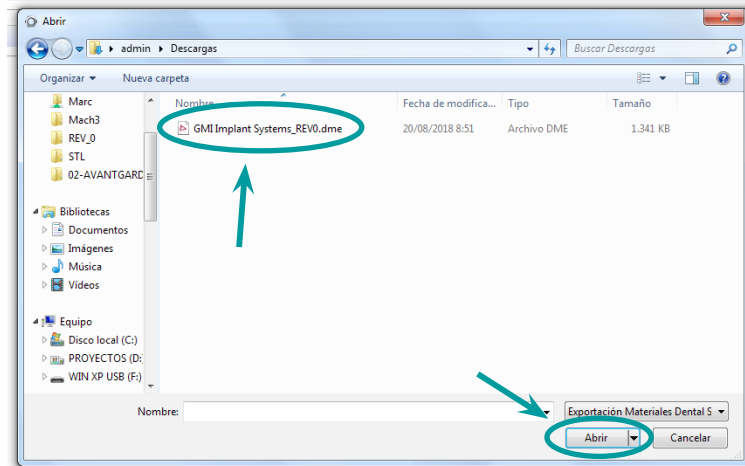
A window will appear to choose the file. Choose the library you have downloaded from the web in the *downloads* folder.

Choose "Open".

## Import library

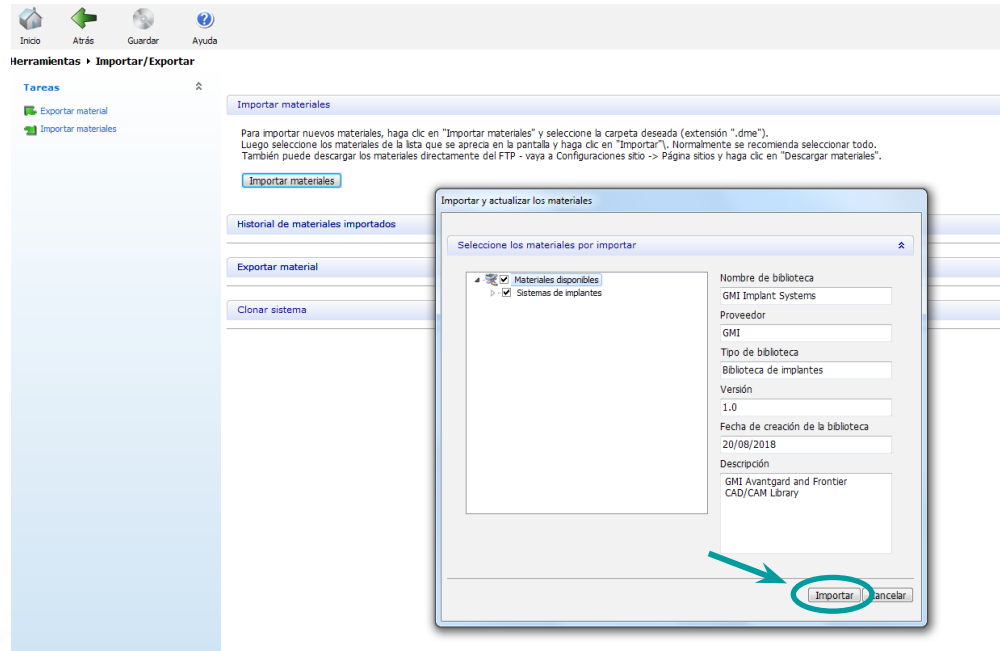
Select "Import".

The screenshot shows the GMI software interface. At the top, there are navigation icons: Inicio, Atrás, Guardar, and Ayuda. Below them is the 'Herramientas' menu with 'Importar/Exportar' selected. On the left, there is a sidebar with 'Tareas' including 'Exportar material' and 'Importar materiales'. The main area displays the 'Importar materiales' dialog box. The dialog box has a title bar 'Importar y actualizar los materiales' and a subtitle 'Selección de los materiales por importar'. It contains a tree view with 'Materiales disponibles' and 'Sistemas de implantes' (checked). To the right, there are input fields for: Nombre de biblioteca (GMI Implant Systems), Proveedor (GMI), Tipo de biblioteca (Biblioteca de implantes), Versión (1.0), Fecha de creación de la biblioteca (20/08/2018), and Descripción (GMI Avantgard and Frontier CAD/CAM Library). At the bottom right, there are 'Importar' and 'Cancelar' buttons. A red arrow points to the 'Importar' button, which is circled in red.


## Import library

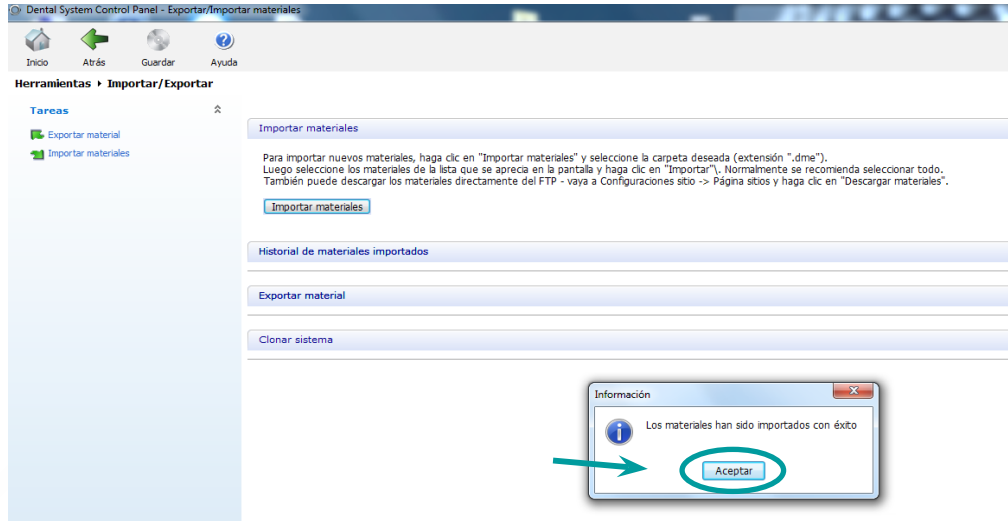Confirm it has been imported correctly.

## Import library

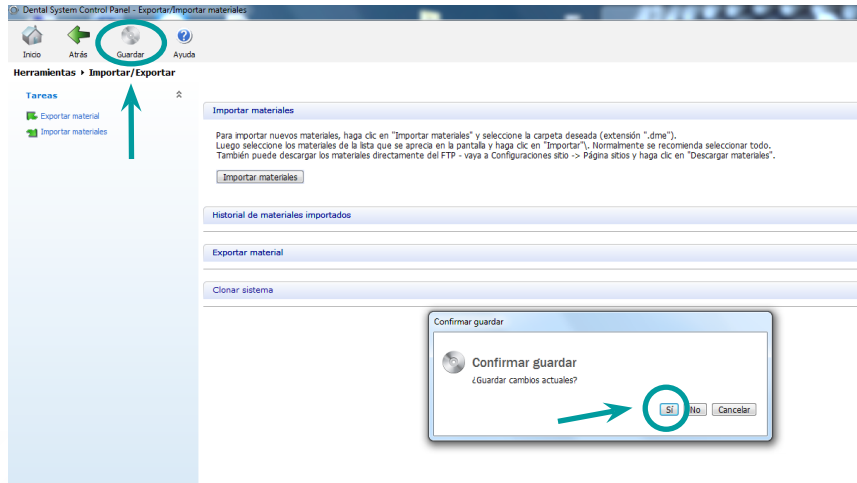Save, confirm and you can use the **gmi** libraries.

Thank you for trusting us